

# Rotary Master+

## Connecting Your Components

*Adding an Electric System has never been so easy!*

### Components




Power cord

Power supply brick and attached cable



Motor control unit and hose

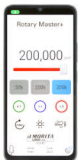

Tablet with installed software



Communication cable

**1** Connect the **Power Supply Brick** to the **Power Cord**, then plug it into a wall outlet.


**2** Insert the power plug into the **Motor Control Unit**.

**3** Attach the 4-hole hose to the **Motor Control Unit**.



**4** Plug **Communication Cable (A)** into the **Motor Control Unit** and connect the other end to the **Tablet (B)**. Turn the Tablet ON.

**5** The Rotary Master+ supports electric attachments from all major brands. Connect your handpiece and begin operation.

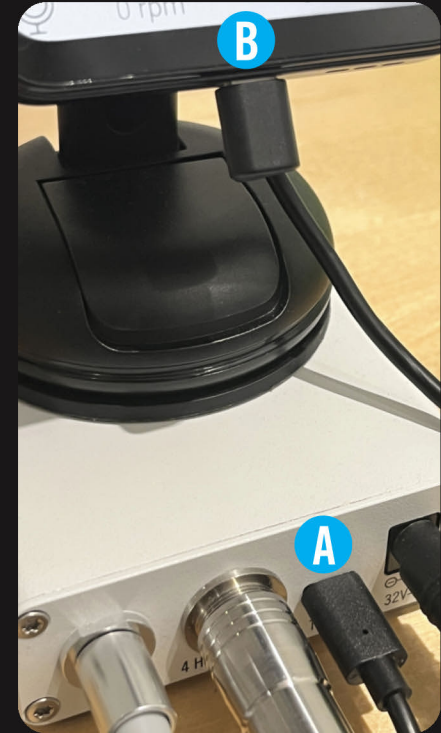

**J.MORITA**  
USA INC.

Manufactured for: J. Morita USA, Inc., 9 Mason, Irvine, California 92618, USA