


1 Turn Tri Auto mini on: Press Main switch.

The standby display will appear.

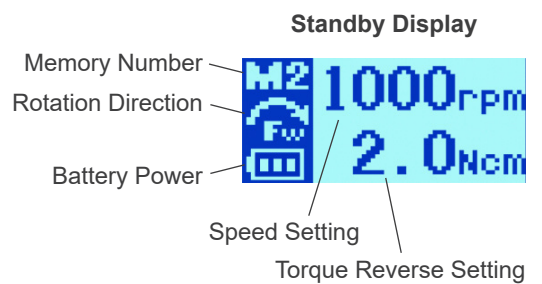

2 Select Memory number: Press Plus or Minus switch.

* There are six memories

The factory settings are shown below.


Setting	Memory		
	M1, M4	M2, M5	M3, M6
Speed (rpm)	400	400	250
Torque Reverse (Ncm)	1.0	0.8	0.6

3 Start motor: Press Main switch again.

The Torque Display will appear.

* If you hold the Main switch down, the motor will run only while it is being held down and stop when you release it.

Torque Display

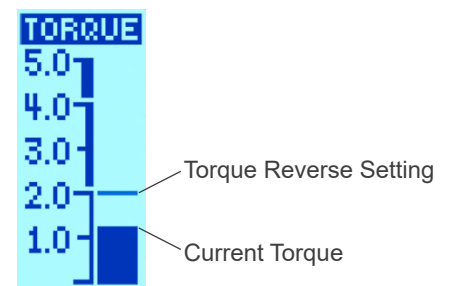

4 Stop motor: Press Main switch again.

The standby display will reappear.

