

Introduction

PowerTemp is a user-friendly temporary cement available in a 50ml cartridge. It is suggested for the cementation of temporary crowns or bridges, as well as try-in and evaluation of permanent restorations. PowerTemp demonstrates a powerful bond to crown and bridge materials, and provides an effective marginal seal, yet remains easy to remove. Its paste-to-paste formula is dispensed through an auto-mix delivery system, and can be used with a conventional dispensing gun and mixing tips, for a direct, nearly effortless application. PowerTemp's non-eugenol and ZnO/Organic based formula will not inhibit the curing of resin-based materials and is an excellent choice when allergies to eugenol are a concern.



Kit contents

50ml (85g) 1:1 Automix Cartridge x 1
Mixing tip (XXS) x 10

Instructions for use

1. Place the cartridge into the dispensing gun retainer. Remove the protective cap from the cartridge channel openings.
2. Test the flow of material from both cartridge channels by compressing the dispensing gun handle. When material flows evenly from both channels, wipe excess material from the cartridge opening with a paper towel or tissue.
3. Match the notch on the mixing tip to the groove on the cartridge. Push and rotate the tip clockwise 90 degrees until it is locked into position.
4. PowerTemp can be applied directly into the temporary crown or bridge.
5. Seat the temporary crown or bridge using thin layer firmly over the prepared tooth structures within 90 seconds.
6. Once the procedure is complete, leave the mixing tip in place to seal the syringe until next use.
7. PowerTemp sets in the mouth within 2-3 minutes.
8. After PowerTemp sets, remove excess material.

WARNINGS

1. Do not re-use the cap. Cross contamination may occur resulting in non-performance of the material.
2. Do not dry prepped teeth, it is best to moisten them with water. A moistened tooth facilitates removing the temporary crown or bridge later on.

Storage

Do not store above 23°C (73° F). Do not use after expiration date.

Manufactured for
J. Morita USA, Inc.
9 Mason, Irvine, CA 92618
info@jmoritausa.com
www.morita.com/usa