


**1 Turn Tri Auto mini on: Press Main switch.**  
The standby display will appear.

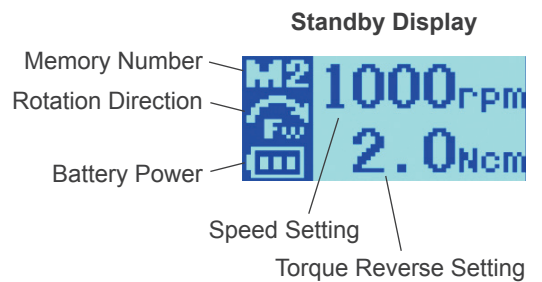

**2 Select Memory number: Press Plus or Minus switch.**  
\* There are six memories


The factory settings are shown below.

Setting	Memory		
	M1, M4	M2, M5	M3, M6
Speed (rpm)	400	400	250
Torque Reverse (Ncm)	1.0	0.8	0.6

**3 Start motor: Press Main switch again.**  
The Torque Display will appear.

\* If you hold the Main switch down, the motor will run only while it is being held down and stop when you release it.

Torque Display

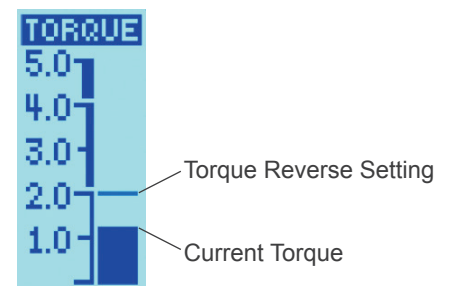

**4 Stop motor: Press Main switch again.**  
The standby display will reappear.

